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REAL CARE CONTAINER FOR THE COLLECTION AND TRANSPORTATION OF STOOL

Sterilized container for the collection and transportation of stool, for analysis and stool culture.

Made from polypropylene, with screw lid for safe transport. Capacity 60ml. Includes spatula for sample collection.

Instructions for use: Remove the container from the sterilized packaging. Unscrew the lid. Using the included spatula, collect the stool sample inside the container. Close securely with the lid. Consult your healthcare provider for the appropriate method of sample collection.

Cautions:

- Do not use if the sterilized packaging is open or if the container lid is open or if the container is damaged.
- In case of any serious adverse event, please report to the Manufacturer and the Competent Authorities.

Symbol Descriptions

	Manufacturer		Date of manufacture
	Batch code		In vitro diagnostic medical device
	Keep away from sunlight		Keep dry
	Consult instructions for use		Unique device identifier
	Sterilized using ethylene oxide		Do not use if package is damaged and consult instructions for use
	Use-by date		Do not re-use
	CE mark		

PROMIFAR
28 Meg. Alexandrou str., Acharne,
13671
T: +30 210 2771398, F +30 210 2409134
E-mail: info@promifar.gr

**Exclusively
distributed
and traded
by:**

PANSYFA A.E.
3-5 Ethnikis Antistaseos Str, 15344,
Gerakas, Attica, Greece
T: +30 210 9645223
E-mail: info@pansyfa.com ,
<https://pansyfa.coop>