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REAL CARE CONTAINER FOR 24HOUR-COLLECTION OF URINE AND TRANSPORTATION

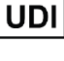
Container for 24hour-collection of urine and transportation. Made from non-toxic, high-density polyethylene. Capacity of 2.5 litres. With two screw lids made of polypropylene - one with a wide opening for urine sample collection and a smaller one for taking the urine sample. Ergonomic design with carrying handle for ease of use.

Instructions for use: Remove the container from the packaging. Unscrew the largest lid. Collect the urine sample inside the container, according to the instructions given by the healthcare professional. After each use, close securely with the lid.

Cautions:

- Do not use if the container lids are open, or do not fit well or if the container is damaged.
- In case of any serious adverse event, please report to the Manufacturer and the Competent Authorities.

Symbol Descriptions

	Manufacturer		Date of manufacture
	Batch code		In vitro diagnostic medical device
	Keep away from sunlight		Keep dry
	Consult instructions for use		Unique device identifier
	Single patient multiple use		Recycling (High Density Polyethylene)
	CE mark		Recycling (Polypropylene)

PROMIFAR
28 Meg.Alexandrou str., Acharne,
13671
T: +30 210 2771398, F +30 210 2409134
E-mail: info@promifar.gr

**Exclusively
distributed
and traded
by:**

PANSYFA A.E.
3-5 Ethnikis Antistaseos Str, 15344,
Gerakas, Attica, Greece
T: +30 210 9645223
E-mail: info@pansyfa.com ,
<https://pansyfa.coop>