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## REAL CARE MEN'S URINE BOTTLE 800ml

**Multiple use men's urine bottle for urine collection.** With safety screw lid. Made of non-toxic, durable, high strength polypropylene. With smooth, round surfaces to prevent injury. With a large opening at the top to receive the sample (urine).

**Instructions for use:** Remove the urine bottle from the plastic packaging. Unscrew the lid. Collect the urine inside the container and secure again with the lid. To reuse the urine container, follow the cleaning instructions.

### Cleaning instructions:

- Use the necessary personal protective equipment (e.g., disposable gloves) to limit possible microbial transfer.
- Empty the contents of the urine bottle into the toilet or drainage system.
- Rinse the urine bottle with warm water, removing any urine residue.
- Apply a mild disinfecting cleaning agent to all surfaces of the urine bottle (internal and external), and clean with a soft brush or sponge.
- Rinse thoroughly with warm water and make sure the urine bottle is clean.
- Allow the urine bottle to air-dry completely or dry thoroughly using a soft, disposable cloth.
- Dispose of personal protective equipment in accordance with national legislation.
- Wash your hands thoroughly with soap and warm water, dry them with a paper towel and dispose of them in a closed bin.

**Storage instructions:** When not in use, the product is recommended to be stored clean, in a cool, dry, well-ventilated environment and away from exposure to direct sunlight, heat, flames, ignition sources and oxidizing agents.

### Warnings:

- Replace the product if it is scratched/stained or has other visible signs of wear or the lid is ill-fitting.
- Discard the product in accordance with local and national regulations.
- In case of serious incidents, report to the Manufacturer and the Competent Authority.

### Symbol Descriptions

	Manufacturer		Date of manufacture
	Batch code		Medical Device
	Keep away from sunlight		Keep dry
	Consult instructions for use		Unique device identifier
	CE mark		Recycling (polypropylene)

PROMIFAR  
28 Meg.Alexandrou str., Acharne,  
13671  
T: +30 210 2771398, F +30 210 2409134  
E-mail: info@promifar.gr

**Exclusively  
distributed  
and traded  
by:**

PANSYFA A.E.  
3-5 Ethnikis Antistaseos Str, 15344,  
Gerakas, Attica, Greece  
T: +30 210 9645223  
E-mail: info@pansyfa.com ,  
<https://pansyfa.coop>